



**MEDLINE INDUSTRIES, LP**

**IMMEDIATE ACTION REQUIRED**

**URGENT MEDICAL DEVICE RECALL**

Dear Medline Customer:

The purpose of this communication is to provide important information including recall instructions related to Namic Angiographic Control Syringes with Rotating Adaptor (“Namic RA Syringes”). Medline Industries, LP (“Medline”) has identified through post-market surveillance a potential risk of the syringe rotating adaptor unwinding during use, which may result in a loose connection and/or full disconnection between the syringe and manifold. Details are included within this communication.

**Risks Associated with Syringe Disconnection and Loose Connection**

This communication is applicable to Namic RA Syringes, which may be purchased individually or as part of Fluid Management Kits.

Through customer complaint data, Medline identified a potential hazard involving the syringe. There have been instances reported of the syringe unwinding itself from the manifold, resulting in the connection becoming loose and/or fully disconnecting. If unwinding occurs, there is potential for biohazard exposure, blood loss, and infection, as well as introduction of air into the line, which may result in air embolism – a condition that carries a risk of serious injury or death. All units have the potential to exhibit this failure mode. There have been three instances reported of inadvertent air injection into the patient, and one instance of exposure to biohazard. None of these instances have resulted in death.

Due to the risk of serious injury or death, customers are directed to remove and destroy all Namic RA Syringes. The only exception is in rare circumstances where angiography is urgently required, no alternative device is available, and postponing the procedure would place the patient at significant risk.

If use is unavoidable because failure to proceed would result in patient harm, the Namic RA syringe must be used with extreme caution and vigilance, including manual stabilization of syringe-to-manifold connections and continuous, 100% monitoring during setup and throughout the entire procedure. In such cases, strict adherence to all usage guidelines and the Instructions for Use is also required, including:

- Ensure that you are making secure connections when using this device to prevent the introduction of air into the system that could result in embolism and in rare instances death.
- All connections should be finger tightened. Over tightening can cause cracks and leaks to occur that could result in embolism and or exposure to biohazards.



- Verify that there is not leakage by aspirating and checking to see that air is not trapped in the system to minimize the potential for embolism and in rare instances death.
- Examine product carefully for entrapped air and fully debubble prior to injection to minimize the potential for embolism and in rare instances death.



A full copy of the IFU can be accessed via the QR code.

In addition to ensuring strict adherence to the IFU, if the syringe unwinds causing loose connection or full disconnection **during procedural set up**, please take the following steps:

1. Remove and replace the syringe.
2. Follow normal use guidelines, in accordance with the Instructions for Use, and proceed with the procedural set up. Finger tighten the syringe and manifold connection. Do not overtighten.
3. Verify the syringe connection remains secure.
4. **Ensure manifold system is flushed and de-bubbled prior to use.**
5. If the syringe replacement continues to show evidence of loosening or disconnecting from the manifold system, **DO NOT USE**. Replace the manifold or entire manifold system.

If the syringe unwinds causing loose connection or full disconnection **during the procedure**, please take the following steps:

1. **Disconnect the manifold system from the patient.**
2. Once the manifold system is disconnected, remove and replace the control syringe.
3. Finger tighten the syringe and manifold connection. Do not overtighten.
4. **Reprime the manifold system to ensure all air has been removed prior to use.**
5. Verify the control syringe connection remains secure without evidence of loosening or disconnecting off the manifold during repriming of the manifold system.
6. Resume procedure with standard catheter connection protocol.
7. In the event the control syringe replacement continues to show evidence of loosening or backs off the manifold system **DO NOT USE**. Replace the entire manifold system.

Sincerely,

Medline Quality Assurance Team



Medline Industries, LP  
**IMMEDIATE ACTION REQUIRED**

Namic Angiographic Rotating Adaptor (RA) Control  
Syringes(Namic RA Syringes)

02/27/2026

Dear Valued Medline Customer:

Medline Industries, LP is issuing a recall for specific item(s) and lot(s) of Namic Angiographic Rotating Adaptor (RA) Control Syringes (Namic RA Syringes). Please refer to the attached document titled R-26-025 Namic RA Syringe Recall Notice for further information and required actions. Please refer to the enclosed list for all affected item number(s) and affected lot number(s).

Adverse reactions or quality problems experienced with the use of this product may be reported to the FDA's MedWatch Adverse Event Reporting program either online, by regular mail or by fax and to Medline Industries, LP.

**REQUIRED ACTION:**

1. Immediately check your stock for the affected item number(s) and the affected lot number(s) listed on the enclosed response form. Quarantine all affected product. Upon completion of the below actions, please destroy affected product.
2. Please complete and return the enclosed response form listing the quantity of affected product you have in inventory. Even if you do not have any affected product in inventory, please complete and return the response form.
3. Your account will receive credit once the response form is submitted, if applicable.
4. If you are a distributor, or have resold or transferred this product to another company or individual, you are required by FDA regulations to notify them of this recall communication. Please request that customers destroy any affected product. You should include your customers' quantities on your response form.

If you have any questions, contact the Recall Department at 866-359-1704 or [recalls@medline.com](mailto:recalls@medline.com).

Please accept our sincere apologies for any inconvenience this may have caused. We, like you, place the health and safety of your patients first and foremost.

Sincerely,

*Karin Johnson*

Karin Johnson  
Product Recall Specialist Sr.

**Enclosed:** Response Form

Recall Ref#: R-26-025

Confidential – For intended recipients only. Do not Distribute.

**Medline Industries, LP.** Three Lakes Drive      Toll Free: 866.359.1704      Email:  
Northfield, IL 60093-2753      Website: medline.com      [Recalls@Medline.com](mailto:Recalls@Medline.com)  
Fax: 866.767.1290



# URGENT ACTION RESPONSE FORM

**Recall Ref#: R-26-025**

C-672  
 Account Number  
 H&A  
 Account Name  
 CT-05-11, Corporate Tower Subang Square  
 Account Street Address  
 Subang Jaya, Selangor, Malaysia, 47500  
 City State Zip

02/27/2026

## MEDLINE INDUSTRIES, LP Namic Angiographic Rotating Adaptor (RA) Control Syringes (Namic RA Syringes)

**Medline Industries, LP cannot replace, ship back, or provide a credit for any product that is not impacted by this voluntary action.**

Item Number (See attached List) <small>(used to purchase)</small>	Lot Number(s) Affected (See Attached list)	Individual Quantity Destroyed*
70095107	125011	235 EA
		EA
		EA

\*Indicate zero "0" if you do not have any lots and/or items in inventory.

**If you have transferred possession of this product to another individual, to another department or another location in your organization you are required by FDA regulations to notify them of this communication.**

By signing below you are verifying that you have received, read, understood and performed any actions specified in the recall letter.

NUR IZZATY  
 \_\_\_\_\_  
**Print Name**  
 03-5636 6732  
 \_\_\_\_\_  
**Telephone Number**



\_\_\_\_\_  
**Signature**  
 1/4/2026  
 \_\_\_\_\_  
**Date**  
 izzaty@hnamedical.com  
 \_\_\_\_\_  
**Email Address**

**Even if you have no affected stock:**  
 Please return the completed form via email to  
[recalls@medline.com](mailto:recalls@medline.com) or fax 866-767-1290

Confidential – For intended recipients only. Do not Distribute.

**Medline Industries, LP.** Three Lakes Drive Toll Free: 866.359.1704 Email:  
 Northfield, IL 60093-2753 Website: medline.com [Recalls@Medline.com](mailto:Recalls@Medline.com)  
 Fax: 866.767.1290



### Item and Lot Numbers

Item Number	Lot Number
80087007	69251, 69917, 69924, 69955, 72771, 77785, 77961, 80251, 80640, 81441, 81509, 82978, 83583, 84511, 85710, 86143, 87262, 88591, 89167, 89223, 89900, 90951, 92215, 92427, 93891, 94516, 94933, 95447, 95467, 96380, 97051, 99609, 99610, 103018, 103037, 103071, 106714, 107645, 108268, 108568, 109958, 110755, 110822, 111582, 114281, 114307, 115713, 115967, 117330, 117797, 118671, 119940, 120026, 120051, 120053, 120316, 120975, 121434, 122584, 122603, 122604, 124680, 124663, 125249, 125286, 126308, 126585, 126586, 126556, 128344, 127955, 129499, 129498, 130238, 132000, 132003, 132374, 132699, 133607, 133695, 134502, 134500, 134897, 135316, 135265, 137719, 137820, 139116, 139172, 140262, 140397, 140966, 140962, 141839, 141861, 142869, 142855, 143993, 143982, 144660, 145515, 145586, 145926, 146475, 146982, 146986, 147169, 147982, 148028, 148577, 149318, 149312, 149583, 150969, 151045, 151063, 151741, 151761, 153191, 153280, 154130, 154083, 154176, 155027, 155028, 155581, 156012, 156816, 157315, 157314, 158415, 159560, 159582, 159552, 159978, 160369, 162481, 162480, 162625, 162624, 162471, 162649, 164098, 164096, 164092, 164097, 164356, 164291, 164412, 164369, 164549, 165293, 165504, 165591, 165654, 166238, 166269, 166502, 167633, 167696, 167868, 167920, 167921, 167958, 167971, 169151, 170064, 170159, 170157, 170191, 170618, 171787, 171814, 171818, 172763, 172766, 173232, 173247, 173231, 173234, 173233, 174297, 174299, 174312, 174326, 174327, 175651, 176231, 176507, 176617, 177589, 177957, 178097, 178099, 178800, 179118, 179119, 179357, 179358, 179359, 179906, 179922, 180197, 181163, 181815, 182069, 182717, 182777, 182776, 182835, 183532, 183531, 183591, 183662, 184332, 184178, 184146, 184352, 184776, 184777, 185118, 185117, 185661, 185660, 186183, 186254, 186212, 186265, 186566, 186876, 187378, 187393, 187532, 187533, 187581, 187583, 187602, 187628, 188843, 188889, 188844, 189198, 189242, 189686, 189684, 189685

Confidential - For intended recipients only. Do not Distribute.



80085007	69912, 70209, 70784, 70861, 71257, 71264, 71267, 72311, 73893, 74209, 74674, 74673, 75332, 75764, 77366, 77812, 79410, 80615, 80614, 80915, 81405, 81771, 83753, 84309, 85648, 85601, 85686, 86214, 86721, 86834, 87070, 88657, 88596, 89423, 90105, 89903, 94136, 94420, 96230, 99930, 102996, 102993, 102995, 102515, 103555, 103488, 106721, 107510, 107536, 106964, 107959, 108050, 108049, 109430, 109592, 110369, 110800, 110820, 110918, 111345, 112227, 112968, 114117, 115130, 115122, 115133, 115715, 116669, 117287, 117333, 117592, 119132, 119131, 120378, 120359, 120379, 120477, 120926, 121799, 121795, 121856, 121863, 122659, 123853, 123896, 124655, 125050, 126183, 126182, 126184, 126181, 127096, 127134, 130082, 130189, 130543, 130888, 131003, 131298, 132780, 132835, 133210, 134869, 135426, 135751, 135757, 137216, 137259, 139521, 140407, 140436, 140460, 141378, 142312, 143949, 144525, 144816, 144858, 146335, 146688, 147279, 147758, 148526, 148881, 149157, 149356, 149534, 149591, 150043, 150047, 150277, 150927, 150974, 151304, 151311, 151562, 151735, 153278, 153284, 154621, 154788, 155411, 155753, 156008, 156801, 157245, 157352, 157585, 157358, 157587, 158717, 159713, 159723, 159984, 160326, 160371, 161181, 162113, 162661, 162771, 162717, 162801, 164285, 164302, 164301, 165507, 165537, 166941, 167627, 168370, 168574, 170238, 170490, 170658, 170657, 170774, 172253, 172184, 172513, 172515, 172514, 172517, 172518, 173418, 173451, 173434, 174053, 174052, 174725, 174726, 174727, 175214, 175215, 175384, 175424, 175529, 176612, 176735, 176730, 176752, 177132, 177401, 177400, 177413, 177961, 178119, 178736, 179115, 179339, 179394, 179845, 179890, 179892, 179934, 180117, 180103, 180167, 181515, 181513, 181916, 182018, 181778, 182911, 182891, 183052, 183051, 183346, 183327, 183527, 184160, 184623, 185648, 185644, 185650, 185749, 185759, 185769, 186950, 186949, 187389, 187394, 187519, 187577, 187576, 187578, 187579, 187575, 188490, 188841, 188887, 188910, 189155, 189196, 189197, 189112, 189111
80075027	73773, 75166, 77039, 88354, 90588, 95298, 98304, 103809, 116987, 117782, 121306, 125285, 125804, 126247, 131716, 132243, 134967, 135264, 137714, 141733, 144473, 147785, 149296, 169772, 175145, 176267, 181484, 188304, 189612, 190429, 192341
70095007	74195, 77218, 78891, 83117, 84495, 86795, 88301, 89971, 96912, 109777, 114876, 118512, 118674, 120001, 120780, 121233, 121698, 122711, 128697, 131996, 135597, 137025, 138175, 140916, 147679, 150532, 159176, 161098, 161137, 166266, 167160, 170152
70085007	74193, 77204, 77974, 79512, 83156, 85956, 86070, 86483, 90174, 90816, 90917, 92492, 94095, 96914, 98330, 99425, 100711, 100826, 101994, 113588, 116407, 116708, 117321, 119569, 120349, 121439, 121700, 123467, 124590, 126063, 127845, 132989, 134345, 134342, 134741, 136418, 138294, 141336, 147492, 149666, 150547, 152486, 152797, 155541, 156129, 156574, 158566, 164128, 166270, 167880, 167879, 168660, 170215, 170237, 171147, 172275, 173410, 187298, 187638, 187601

Confidential - For intended recipients only. Do not Distribute.



70087007	75190, 75227, 78287, 89093, 90805, 90975, 93951, 94198, 94735, 96910, 101139, 101618, 106005, 105984, 107037, 109850, 110377, 111490, 111537, 112994, 113383, 113248, 113633, 117395, 121309, 122108, 123468, 124591, 127017, 128278, 129370, 130125, 132167, 134842, 138681, 140049, 153343, 156539, 157741, 159414, 166268, 167639, 167626, 169163, 172756, 176061, 178804, 182016, 186160, 187605
80095107	75353, 76807, 77845, 78909, 80922, 82325, 86417, 91879, 92505, 94737, 96686, 97481, 100824, 103008, 104795, 107307, 109907, 112016, 114170, 114589, 116629, 117564, 120313, 120991, 121734, 125013, 126525, 128345, 129428, 131286, 132103, 136454, 139380, 143033, 143195, 144688, 144687, 145834, 150810, 151611, 151728, 153307, 153320, 154923, 154924, 162864, 163305, 163526, 164371, 165157, 166286, 167026, 167219, 169149, 170143, 170163, 175479, 176026, 177587, 177606, 179362, 180730, 181132, 182200, 185658, 190518, 191423
70097107	75365, 79731, 87346, 94790, 100287, 104614, 105597, 109564, 110688, 118408, 119439, 122041, 124165, 125525, 131987, 133030, 137305, 147507, 147678, 153736, 161611, 165564, 181799, 184713, 185058, 190607
70085107	75384, 96807, 133033, 147541, 154518, 167033, 179525
80097107	77040, 103845, 108241, 109897, 112900, 124191, 131659
70083007	77029, 81537, 84335, 87347, 89563, 113792, 115753, 122956, 127785, 131986, 158748, 175535, 187926
70088007	77187, 86247, 90162, 95619, 104749, 109521, 119070, 122098, 125541, 135421, 138529, 141833, 156061, 157744, 161113, 163952, 168023, 172923, 178145, 186934
70075027	77215, 79109, 82289, 84363, 90369, 107045, 114900, 120770, 122275, 125531, 125834, 132127, 132132, 135020, 135023, 137279, 141699, 141698, 144189, 147731, 149644, 151436, 156104, 156186, 156393, 156392, 158599, 159566, 161568, 165825, 167199, 176256, 176749, 187350
70084007	78186, 81539, 84274, 86756, 89554, 94842, 96918, 105613, 109479, 111244, 114562, 115421, 117317, 118251, 120265, 120769, 121714, 123466, 124589, 131984, 131989, 138071, 140322, 142850, 147002, 152069, 162694, 173507, 174662, 174773, 176555, 178806, 189895
70087107	78897, 148131, 157141, 163681, 167174, 168020, 170851, 176075, 182252, 192254
70097007	79553, 87472, 88541, 94831, 131995, 156391, 157135, 162685

Confidential - For intended recipients only. Do not Distribute.

**Medline Industries, LP.** Three Lakes Drive Toll Free: 866.359.1704 Email: [Recalls@Medline.com](mailto:Recalls@Medline.com)  
 Northfield, IL 60093-2753 Website: [medline.com](http://medline.com) Fax: 866.767.1290



80084007	80553, 80919, 82969, 85041, 88044, 94514, 99870, 101882, 102372, 105155, 107328, 109442, 112810, 112966, 114317, 115744, 117572, 118722, 118962, 119950, 120931, 121305, 121393, 122583, 122606, 123102, 126198, 126566, 127132, 130798, 131678, 132838, 137176, 139543, 140507, 142057, 142279, 143110, 143264, 143714, 148872, 150059, 150488, 151054, 151295, 151300, 151768, 152754, 153438, 154148, 156028, 158045, 158122, 159113, 159652, 160479, 160753, 161269, 162090, 162083, 162336, 162895, 164541, 164540, 166504, 166983, 167995, 168000, 169147, 169560, 169793, 170823, 171778, 173402, 174087, 174697, 174747, 176011, 176240, 176850, 177923, 179193, 179414, 180511, 182186, 182782, 182779, 185643, 186234, 186874, 187363, 187362, 188842, 188845, 188837
70084017	84509, 86823, 95687, 165551, 165868, 166300
70095107	88193, 90703, 91441, 108589, 112295, 113957, 114591, 114910, 116253, 117305, 121678, 122871, 125011, 125016, 126248, 132001, 147323, 147489
70098107	95693, 111595, 131985, 159175
80083007	97388, 127072, 135012, 137004
80084017	101535, 103862, 105095, 106954, 108237, 122887, 126590, 127159, 128360, 128852, 132717, 134047, 135616, 140813, 144899, 148530, 149322, 152430, 161279, 164538, 169107, 173193, 173256, 182141, 183619, 187345, 188399
80095007	105201, 106354, 112875, 114278, 123107, 125259, 143729, 145697, 157989, 161262, 162833, 162869, 164539, 172176, 172859, 176097, 178913, 181201, 181321, 185560

Confidential - For intended recipients only. Do not Distribute.